




Cervical Solutions

**Mobi-C<sup>®</sup>**  
Cervical Disc



Mobi-C Cervical Disc is the first FDA approved two-level disc.<sup>1</sup>

## OVER 70,000 MOBI-C DISCS HAVE BEEN IMPLANTED IN 25 COUNTRIES SINCE 2004.

The Mobi-C Cervical Disc is the first cervical disc in the United States approved to treat more than one level of the cervical spine, and was determined by the FDA to be statistically superior to fusion at 7 years for two-level cervical disc replacement, based on the primary study endpoint of a prospective, concurrently controlled and randomized, multi-center clinical trial.<sup>1</sup>

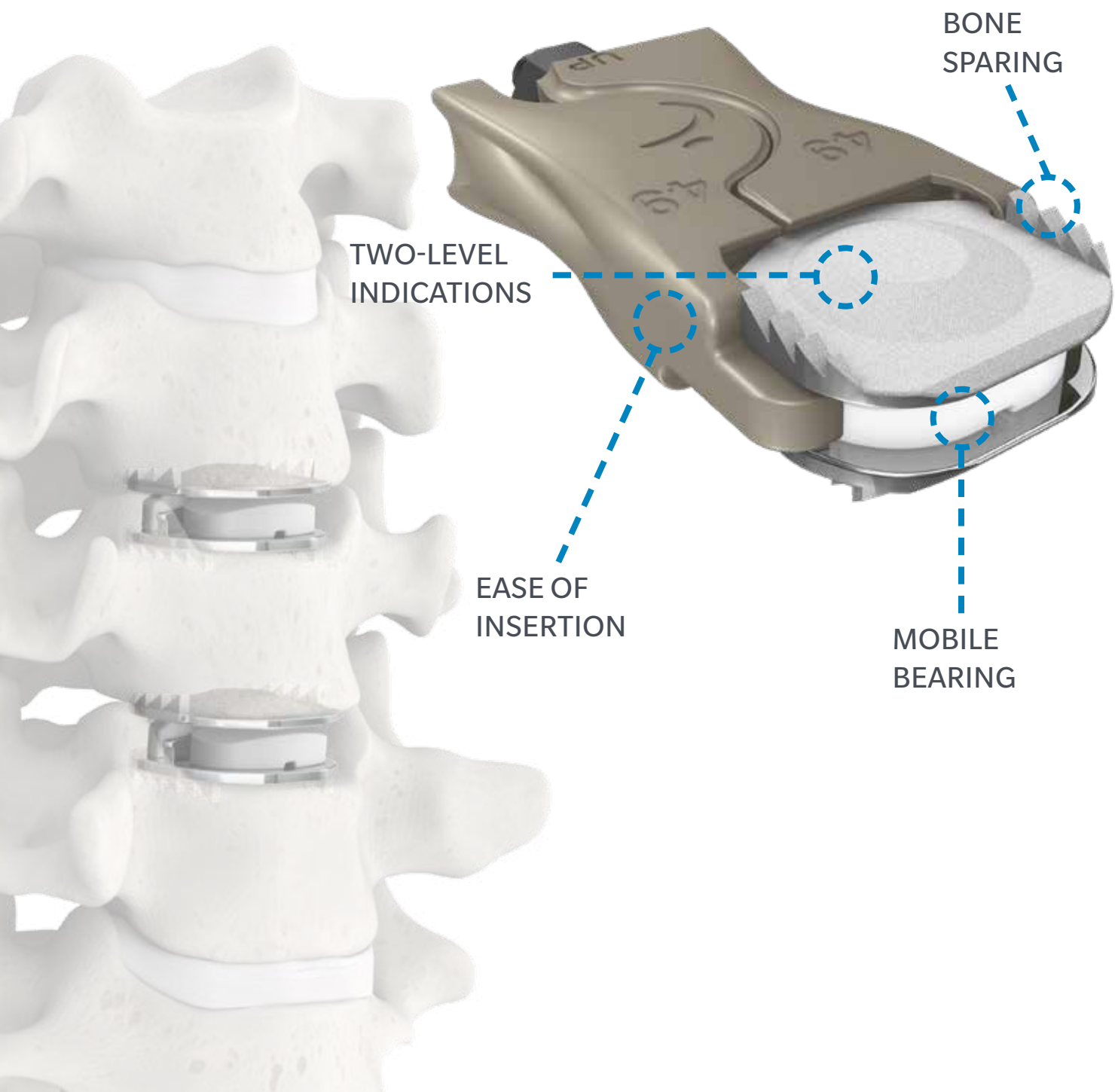

## Mobi-C is non-inferior to ACDF at one level and superior at two levels<sup>2</sup>

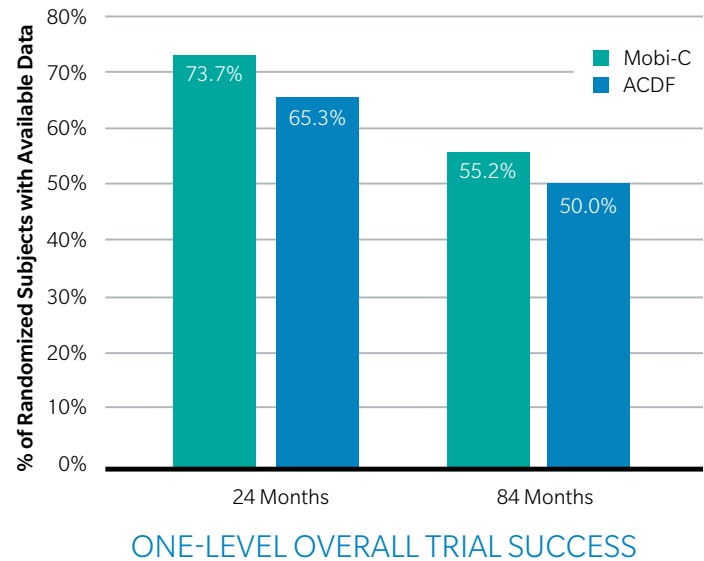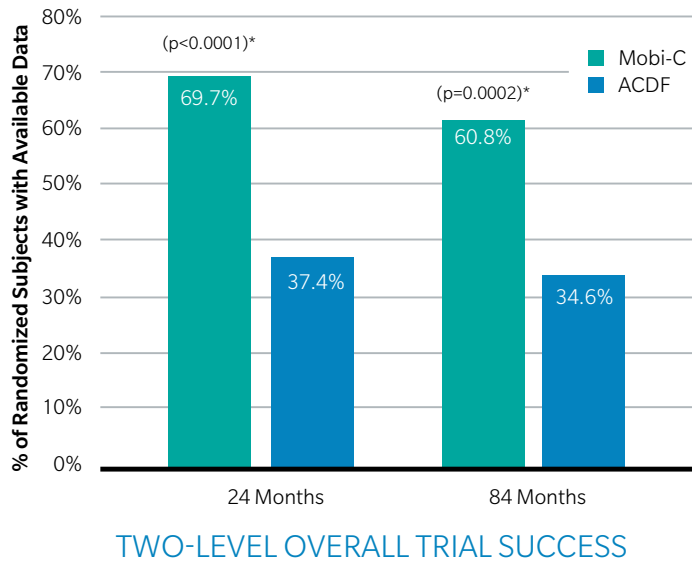

\*Fisher's Exact test used to compare treatments

## Mobi-C had lower rates of adjacent segment degeneration at 84 months

**One-level** deterioration of adjacent segments at 84 months compared to baseline:

- 43.8% for Mobi-C compared to 63.0% for ACDF at the inferior level.
- 40.4% for Mobi-C compared to 65.1% for ACDF at the superior level.

**Two-level** deterioration of adjacent segments at 84 months compared to baseline:

- 30.3% for Mobi-C compared to 66.7% for ACDF at the inferior level.
- 37.5% for Mobi-C compared to 73.7% for ACDF at the superior level.

## Mobi-C had fewer subsequent surgeries at 84 months


**One-level:** Only 3.4% of Mobi-C patients compared to 11.1% of ACDF patients reported secondary surgeries at the index levels through 84 months.

**Two-level:** Only 5.6% of two-level Mobi-C patients compared to 17.1% of ACDF patients reported secondary surgeries at the index levels through 84 months.

**Note:** Success criteria for these Mobi-C Independent Device Exemption (IDE) trial results can be found on page 7.

## MOBI-C: MOBILE BEARING

### Controlled mobility


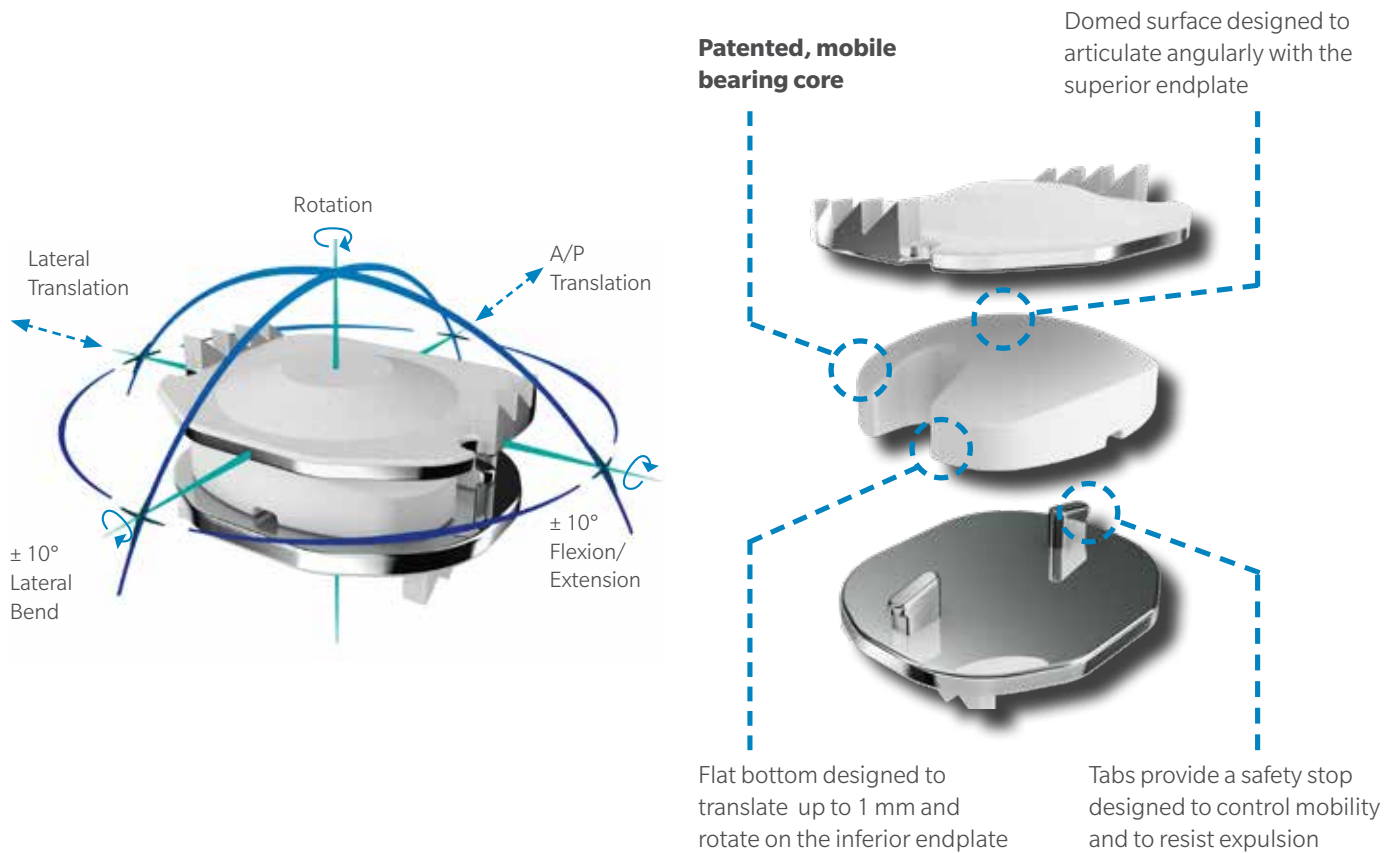


### Self-adjusting

The center of rotation at each level of the cervical spine is variable and constantly changing<sup>3</sup>. Mobi-C was designed to adapt to the Instantaneous Axis of Rotation through its self-adjusting mobile core.

The Mobi-C mobile core:

- Is designed to facilitate independent and coupled motion similar to natural cervical spine motion.
- Moves with the spine and does not dictate a predetermined, fixed axis of rotation.

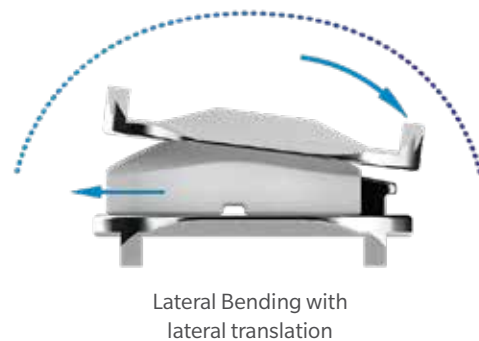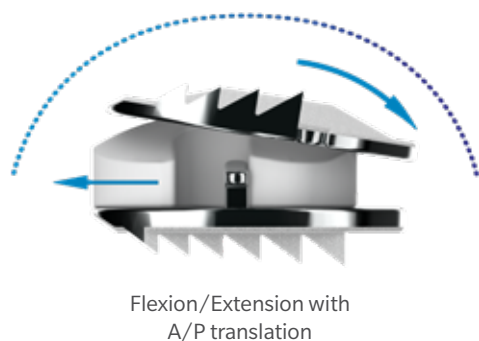

## MOBI-C: BONE SPARING

### No bone chiseling

Mobi-C's mobile core is designed to create low stress at the implant to bone interface. The Mobi-C requires:


- No invasive keels or screws.
- No bone removal for keel preparation.
- No additional operative steps for keel cutting.

Intact endplates, compared to endplates prepared for keels, provide:

- A preserved surface for the implant, ideal for two-level implantation.
- Intraoperative flexibility to optimize implant positioning.



### Short and long term stability

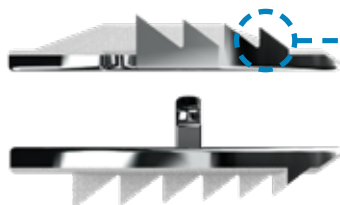


#### Lateral, inclined teeth:

- Purchase in the apophyseal ring to provide initial stability
- Designed to resist migration

#### Superior dome:

- Designed to match the natural, bony anatomy enabling short and long term stability

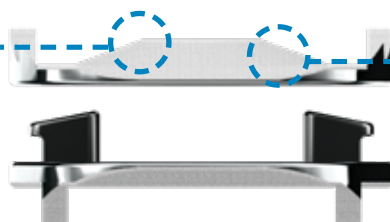

#### Plasma sprayed titanium and hydroxyapatite coated endplates:


- Encourage bony ongrowth for long term stability



## MOBI-C: EASE OF INSERTION

### One-step insertion



- No drilling or chiseling required
- No additional exposure or operative steps required for screw or keel placement

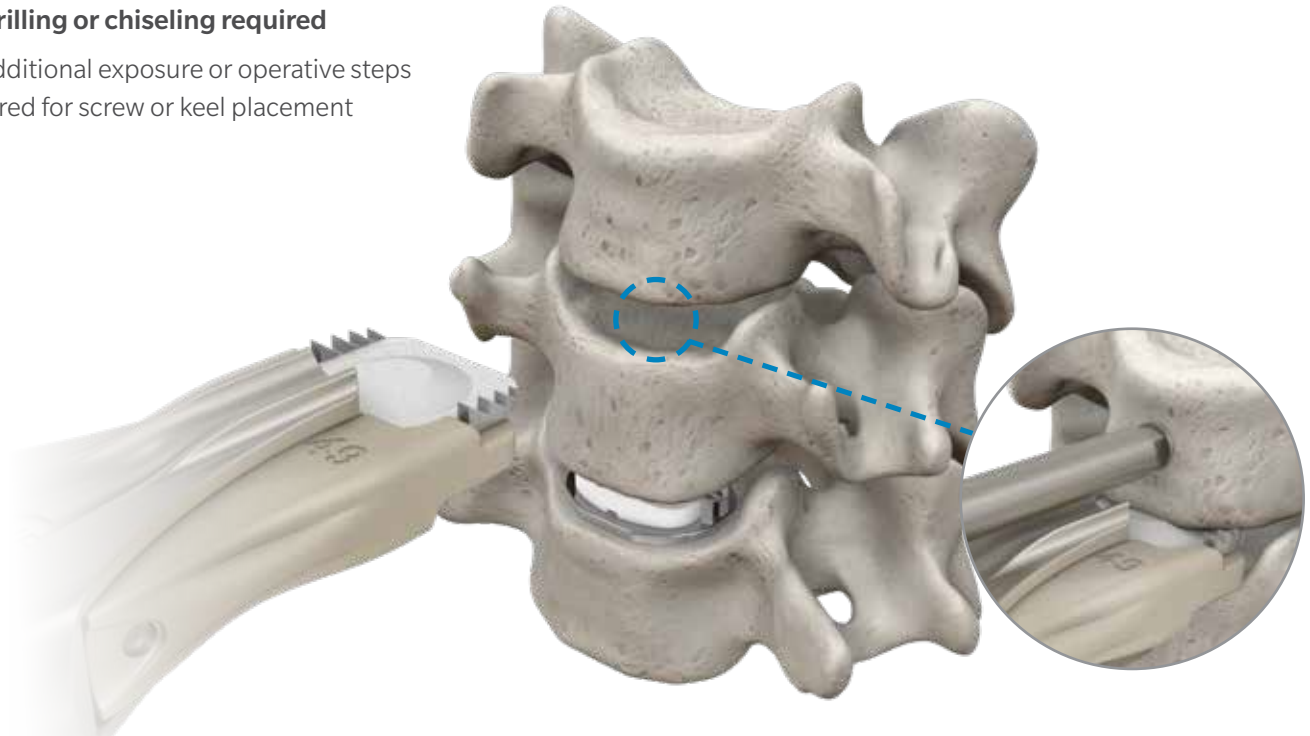

### Pre-assembled implants

The Mobi-C is **delivered pre-assembled** on a disposable PEEK cartridge.

PEEK cartridge assembles easily to the implant inserter saving operative steps

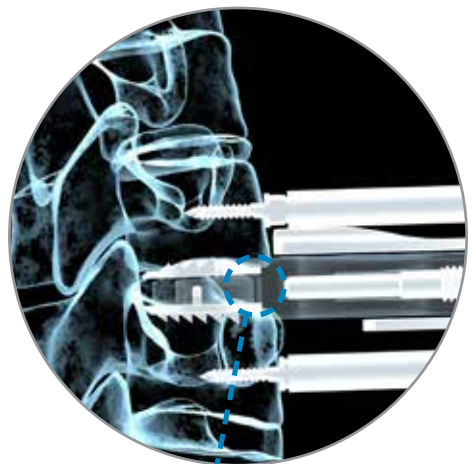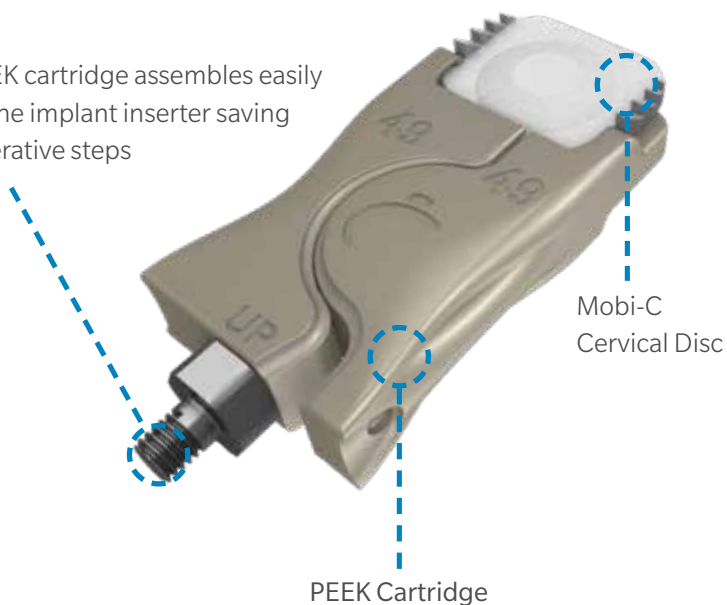

PEEK cartridge allows a radiolucent view of the implant for optimal positioning

## MOBI-C: PRODUCT DETAILS

### Indications

The Mobi-C Cervical Disc Prosthesis is indicated in skeletally mature patients for reconstruction of the disc from C3-C7 following discectomy at one or two contiguous levels for intractable radiculopathy (arm pain and/or a neurological deficit with or without neck pain) or myelopathy due to abnormality localized to the level of the disc space and at least one of the following conditions confirmed by radiographic imaging (CT, MRI, X-rays): herniated nucleus pulposus,

spondylosis (defined by the presence of osteophytes), and/or visible loss of disc height compared to adjacent levels. The Mobi-C Cervical Disc Prosthesis is implanted using an anterior approach. Patients should have failed at least 6 weeks of conservative treatment or demonstrated progressive signs or symptoms despite nonoperative treatment prior to implantation of the Mobi-C Cervical Disc Prosthesis.

### Sizing options

Depth x Width (mm)	Height (mm)
13 x 15	5, 6, and 7
13 x 17	
15 x 15	
15 x 17	
15 x 19	
17 x 17	
17 x 19	



### IDE success criteria

#### Overall Trial Success

Trial success was based on a composite endpoint. A patient was considered a success at 84 months if all of the following criteria were met:

- Sufficient NDI improvement
  - If baseline NDI is  $\geq 30$  points out of 50 points, improvement must be  $\geq 15$  points out of 50 points
  - If baseline NDI is  $< 30$  points out of 50 points, improvement must be  $\geq 50\%$  improvement

*Note: NDI success criteria is measured using a subject answered questionnaire, which assesses the effect of pain on daily life. Each of the 10 assessed criteria receives a score from 0 to 5; the highest score (50 points) represents the most disabled. The final score can be converted to a percentage.*

- No secondary surgery at the treated level
- No radiographic failure
- No neurologic deterioration
- No adverse event determined to be a major complication

#### Adjacent Segment Degeneration

Adjacent segment degeneration was assessed at the spinal segment immediately above and below the treated levels. An independent core lab assessed degeneration using the Kellgren-Lawrence five point grading scale. An adjacent segment was counted as having degenerated if the segment worsened by 1 grade or more.

#### Secondary Surgeries At the Treated Levels

The patient was considered a success in terms of secondary surgery if none of the following were necessary at either of the treated levels: removal, revision, reoperation, or supplemental fixation.

**References:**

1. As of 7/30/13 in the USA
2. The control group in the Mobi-C IDE clinical trial was ACDF using allograft bone and an anterior cervical plate.
3. Amevo B, et al Instantaneous axes of rotation of the typical cervical motion segments: a study in normal volunteers. Clin Biomech (Bristol, Avon). 1991 May;6(2):111-7

800.447.3625 / [zimmerbiomet.com](http://zimmerbiomet.com)

©2018 Zimmer Biomet Spine, Inc. All rights reserved.



**ZIMMER BIOMET**  
Your progress. Our promise.®

All content herein is protected by copyright, trademarks and other intellectual property rights owned by or licensed to Zimmer Biomet Spine, Inc. or one of its affiliates unless otherwise indicated, and must not be redistributed, duplicated or disclosed, in whole or in part, without the express written consent of Zimmer Biomet Spine. This material is intended for health care professionals, the Zimmer Biomet Spine sales force and authorized representatives. Distribution to any other recipient is prohibited.

PEEK CLASSIX is a trademark of InVivo Biomaterial Solutions.

2076.1-US-en-Rev0718