



## NuVasive® Helix Mini ACP™



The premium features you've  
come to expect – made faster,  
smaller, and more secure.

## NuVasive® Helix Mini ACP™ System Overview

### NuVasive Helix Mini ACP

- Fixed- and Variable-Angle Bone Screws for the perfect construct type, depending on patient needs.
- Variable-Angle Bone Screws feature a 20° cone-of-angulation to accommodate surgeon needs.
- NuVasive Helix Mini ACP Plate has a 2.4mm profile to reduce the chance of postoperative discomfort.
- Plate sizes accommodate 1- and 2-Level construct needs.



**UNLOCKED**

Ledge of Bone Screw is visible, and head of Screw is proud to Plate surface.




**LOCKED**

Ledge of Bone Screw is completely visible, and head of Screw is seated below Helix Canted Coil Lock.

### Helix Canted Coil Lock (CCL)

- Helix Canted Coil Lock consistently and securely blocks Bone Screws into the Plate while still allowing the Plate to be securely lagged to bone for a tight fit.
- Bone Screws can be removed and repositioned easily while maintaining CCL integrity.
- Aggressive Bone Screws in a full complement of lengths and diameters accommodate a variety of surgical scenarios.

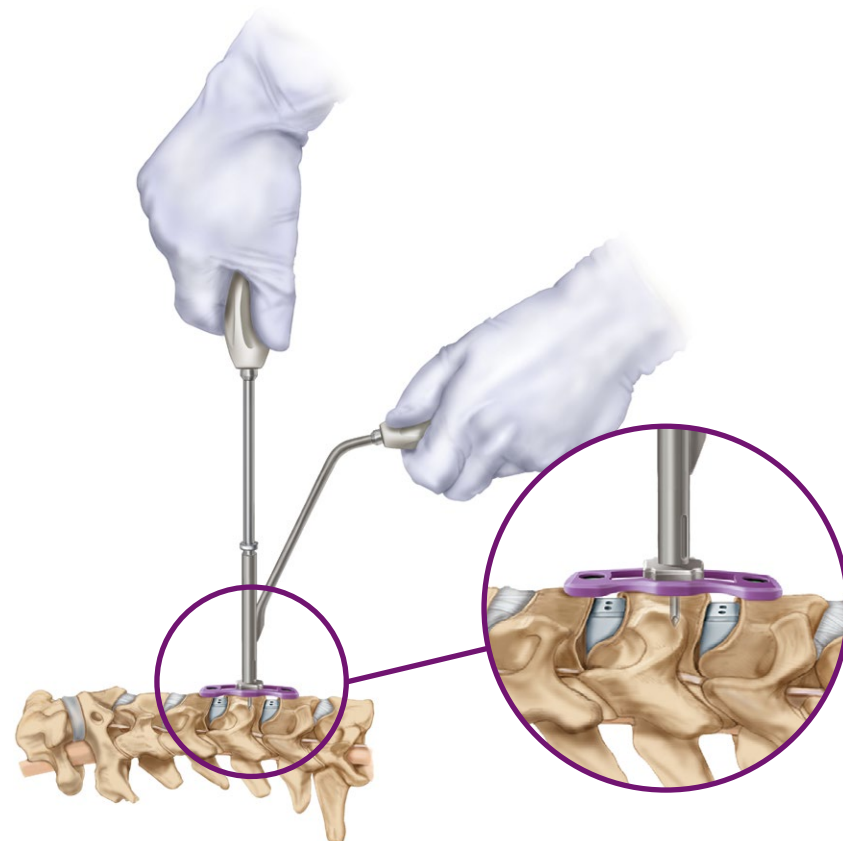


### Balanced and Intuitive Instrumentation

- DTS Guides allow pre-drilling, tapping, and Bone Screw placement without changing insertion angle.

## NuVasive® Helix Mini ACP™ System Overview


### Easy and Precise Intraoperative Plate Bending

- The plate may be bent intraoperatively into either further lordosis or into kyphosis in order to accommodate patient pathology.

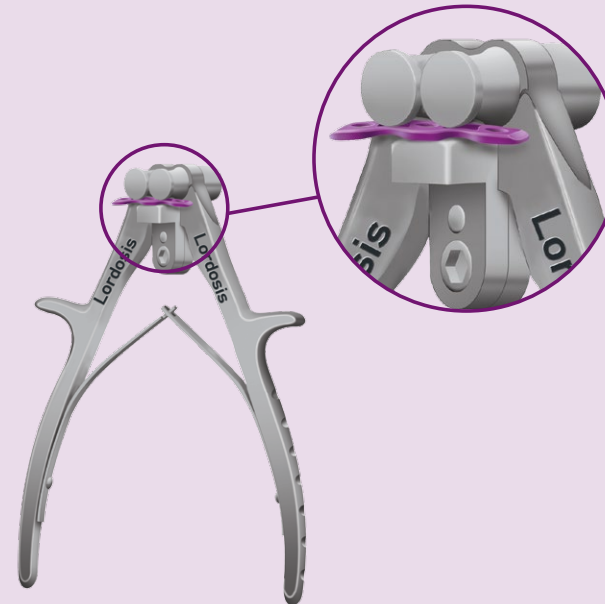


### Narrow Plate Reduces Retraction During Placement

- Reduced retraction for easy placement.
- Less than 12mm at widest point.
  - 10mm at the waist.

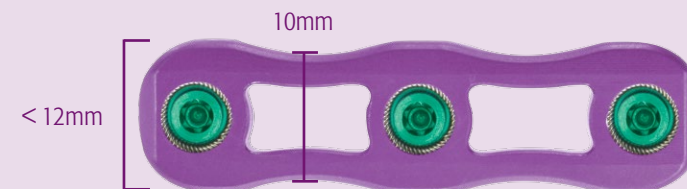

### Low-Profile Leading Edge

- Plate sits flush against anterior cervical spine.
- Decreased “step” on leading edge of Plate to reduce esophageal discomfort postoperatively.

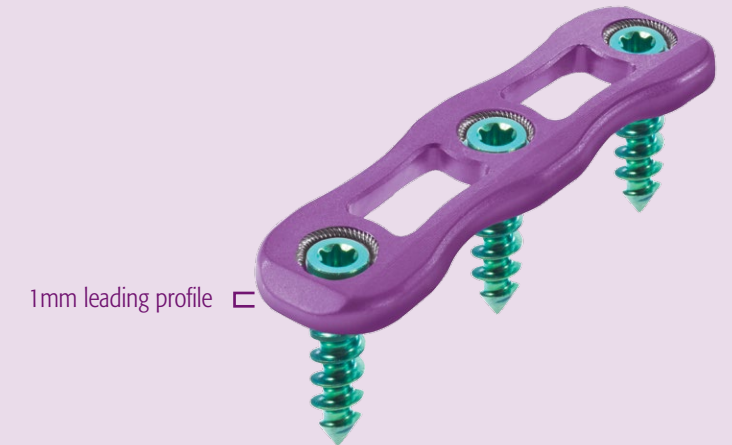

### Intraoperative Visualization

- Large portals allow for graft placement verification.
- Unique graft-viewing/alignment window gives accurate reference for straight midline placement.



## NUVASIVE® HELIX MINI ACP™ IMPLANTS AND INSTRUMENTS


IMPLANTS	CATALOG #
1-Level NuVasive Helix Mini ACP, 20mm	7740120
1-Level NuVasive Helix Mini ACP, 22mm	7740122
1-Level NuVasive Helix Mini ACP, 24mm	7740124
1-Level NuVasive Helix Mini ACP, 26mm	7740126
1-Level NuVasive Helix Mini ACP, 28mm	7740128
1-Level NuVasive Helix Mini ACP, 30mm	7740130
1-Level NuVasive Helix Mini ACP, 32mm	7740132
1-Level NuVasive Helix Mini ACP, 34mm	7740134
2-Level NuVasive Helix Mini ACP, 36mm	7740236
2-Level NuVasive Helix Mini ACP, 38mm	7740238
2-Level NuVasive Helix Mini ACP, 40mm	7740240
2-Level NuVasive Helix Mini ACP, 42mm	7740242
2-Level NuVasive Helix Mini ACP, 44mm	7740244
2-Level NuVasive Helix Mini ACP, 46mm	7740246
2-Level NuVasive Helix Mini ACP, 48mm	7740248
2-Level NuVasive Helix Mini ACP, 50mm	7740250

INSTRUMENTS	CATALOG #
Bone Screw Driver	7730042
Drill Guide Awl	7730062
Self-Centering Awl (Variable)	7730063
Self-Centering Awl (Fixed)	7730064
Bone Screw Extractor	7730072
Drill Guide, Fixed Angle	7730081
Drill Guide, Variable Angle	7730091
Plate Bender	7740701
NuVasive Helix Mini ACP Sterilization Tray	7740001
7° End DTS Guide	7740086
0° Mid DTS Guide	7740088
NuVasive Helix Mini ACP Plate Holder	7740043
Universal Handle	7730600
NuVasive Helix Mini ACP - IFU	9400236
Drill Bit	7730016
Tap	7730026

IMPLANTS	CATALOG #
<b>Self-Drilling Bone Screws (Variable, 4.0)</b>	
4.0 x 11mm Self-Drilling Variable Screw	7741111
4.0 x 13mm Self-Drilling Variable Screw	7741113
4.0 x 15mm Self-Drilling Variable Screw	7741115
<b>Self-Drilling Bone Screws (Fixed, 4.0)</b>	
4.0 x 11mm Self-Drilling Fixed Screw	7742111
4.0 x 13mm Self-Drilling Fixed Screw	7742113
4.0 x 15mm Self-Drilling Fixed Screw	7742115
<b>Self-Tapping Bone Screws (Variable, 4.0)</b>	
4.0 x 11mm Self-Tapping Variable Screw	7748111
4.0 x 13mm Self-Tapping Variable Screw	7748113
4.0 x 15mm Self-Tapping Variable Screw	7748115
<b>Self-Tapping Bone Screws (Fixed, 4.0)</b>	
4.0 x 11mm Self-Tapping Fixed Screw	7747111
4.0 x 13mm Self-Tapping Fixed Screw	7747113
4.0 x 15mm Self-Tapping Fixed Screw	7747115
<b>Self-Tapping Bone Screws (Variable, 4.5)</b>	
4.5 x 11mm Self-Tapping Variable Screw	7735111
4.5 x 13mm Self-Tapping Variable Screw	7735113
4.5 x 15mm Self-Tapping Variable Screw	7735115
<b>Self-Tapping Bone Screws (Fixed, 4.5)</b>	
4.5 x 11mm Self-Tapping Fixed Screw	7746111
4.5 x 13mm Self-Tapping Fixed Screw	7746113
4.5 x 15mm Self-Tapping Fixed Screw	7746115



To order, please contact your NuVasive® Sales Consultant or Customer Service Representative today at:  
**NuVasive, Inc.** 7475 Lusk Blvd., San Diego, CA 92121 USA • phone: 800-475-9131 fax: 800-475-9134  
**NuVasive UK Ltd.** Suite B, Ground Floor, Caspian House, The Waterfront, Elstree, Herts WD6 3BS UK

 phone: +44 (0) 208-238-7850 fax: +44 (0) 207-998-7818

[www.nuvasive.com](http://www.nuvasive.com)

9500188 B

